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C Search

REGISTER

Main ReWrite C.Deletion Start Comparision Result

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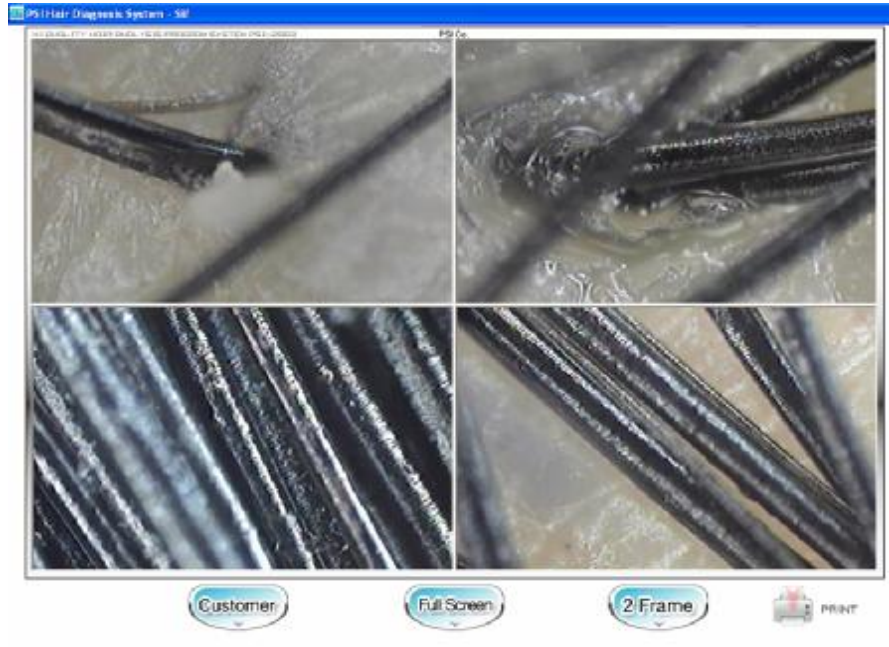
Scalp and Hair Diagnosis 1 Step - Comment

HAIR ANALYSIS SYSTEM

PSI Co. Name : jane kim Sex : Female Age : 58

1. 1st Step for Scalp and Hair Inspection - Questionnaire

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3. Does your hair become glossy in just one day after washing it?

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Scalp and Hair Diagnosis 2 Step - Scalp

HAIR ANALYSIS SYSTEM

PSI Co. Name : jane kim Sex : Female Age : 58

2nd Step for Scalp and Hair Inspection - Scalp Inspection

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HAIR ANALYSIS SYSTEM

PSI Co. Name : jame kim Sex : Female Age : 58

3, 3rd Step for Scalp and Hair Inspection - Hair Inspection

Please answer the questions below, after scrutinizing your hair.

1. Is there hair **damage**?
 Yes No
2. Is there **dandruff** in your hair?
 Yes No
3. Is your hair **glossy**?
 Yes No
4. Is the end of hair **split**?
 Yes No
5. Does **oil** run on the hair?
 Yes No

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Progress of Hair Loss Diagnosis 1 Step - Comment

HAIR ANALYSIS SYSTEM

PSI Co. Name : jame kim Sex : Female Age : 58

1, 1st Step for Hair Loss Inspection - Questionnaire

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10. Have you been exposed to a severe external friction, pressure or stroke, or have you been hospitalized for a long time?

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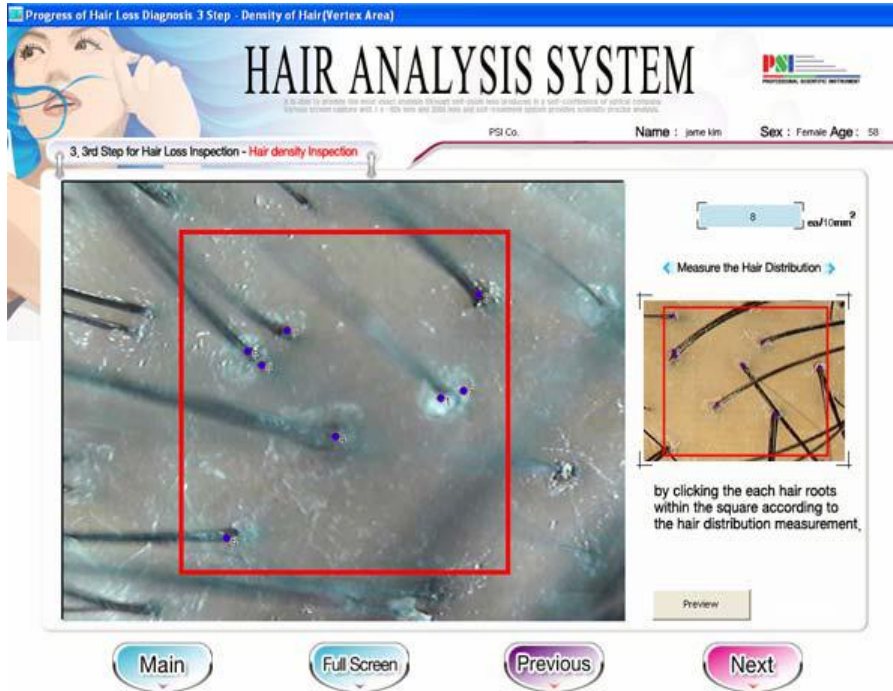
S.I.F
HAIR ANALYSIS SYSTEM
P26 QUALITY CENTER ÇUKUR SİSTEM P26L 2003S

SHOP: PSI Co.
TEL : 0312 555 9999

NAME: Jam es klm SEX: Female AGE: 26 DATE: 2003-7-1 TIME: 1

Result

Scalp hair type I
Normal

Questions I
Injury Hair

Hair loss type
Type 0
STEP 2

Scarring hair loss I
Exist

Progress Probability Hair loss I
Alopecia areat

CUSTOMER IMAGE

Transition Graph

Scalp I Hair I Aroud vertex I Aroud vertex I

Analysis Image

Scalp I Hair I Hair thickness I Hair density I Aroud vertex I

Details

Scalp Hair Type - Normal -
[if you don't care]
If you are in a good state, it is important for you to make efforts to maintaining it. As people become older, the scalp is to becoming dry, and the serious environmental problems in the modern society increase the risk of hair loss. Furthermore, stresses accumulated in human body may cause hair loss to a healthy hair in a moment because stress may cause an impediment of blood circulation.
[Treatment]
In that the hair is also a part of human body, you need proper exercise and good diet to keep your hair healthy. Additionally, you need to keep your scalp and hair clean, and appropriate massage is recommended for smooth blood circulation of the scalp. The capillary receives and sends oxygen and nutrition to the cells, and the former receives carbon dioxide and waste matters from the latter and eliminate them from the system. Accordingly, you need to have your scalp massaged for betterment of blood circulation in the scalp. Also, resolution of stresses by enough relaxation is recommended for smooth blood circulation. Use shampoo products every other day.

Dandruffy Hair - Damaged -
[if you don't care]
Hair damage is caused by (1) hair with inherently low protein, (2) dryness of scalp and hair, (3) exposure to high temperature heat such as dry-cleaning, (4) chemical damage by dyeing or permanent wave, (5) physical friction, (6) use of shampoo in low quality or inappropriate to the hair, or (7) ultraviolet rays due to longstanding exposure to strong sunlight. Accordingly, it is important to identify the features of your hair when dyeing or applying permanent wave or dry cleaning, or choosing shampoo. Greater care should be taken to hair damage. Severely damages hair may need to be cut when it is in a unrecoverable state. Cutting of hair or using special treatment products is needed depending on the extent of damage.

Scarring Hair Loss
Such parts in which pores were destroyed due to wound, scald or infection are not recoverable. Since various surgeries are needed, consultation with experts is required.

Non-scar Hair Loss
Alopecia areat - Its cause is not clear. It is mostly associated with severe stress, abnormality of immune system or disorders such as atopic dermatitis, thyroid gland disorder, leukoplakia, etc.
Efforts to resolve stress and relax yourself and special treatment are needed.

Advice

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Diagnosis Result

MAIN RESULT ADVICE HISTORY PRINT

SIF
HAIR ANALYSIS SYSTEM
HIGH QUALITY HAIR CARE SYSTEM PSI 2003

HAIR ANALYSIS HISTORY

SHOP: PSI Co.
TEL: 031-255-8658

NAME: James kim SEX: Female AGE: 26 DATE: 2003-7-1 TIME: 4

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